



A moist environment has been clinically proven to aid and speed up the natural wound healing process.

PRODUCT BENEFITS:

- ✔ Helps soothe pain
- ✔ Provides barrier against external contamination
- ✔ Reduces the likelihood of scarring
- ✔ Very skin-friendly
- ✔ Non-sticky and non-greasy formulation



Flamigel®

Flen Health
advanced skin healing

Flen Health UK Ltd.
1 Northumberland Avenue, Trafalgar Square,
WC2N 5BW, London United Kingdom
Tel: +44 207 872 5460
www.flenhealth.com

Flamigel®



Essential
IN YOUR
FIRST-AID KIT

Flamigel® is a safe and effective treatment for superficial wounds.

Sunburn



Superficial Burns



Cuts



Grazes




Post laser therapy




4-STEP WOUND CARE with Flamigel®

① CLEANING

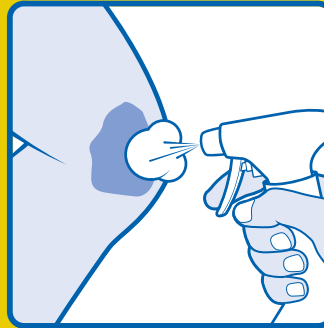


Wash your hands and clean the wound thoroughly. Rinse the wound under running water or use a wound cleanser.

③ TREATMENT

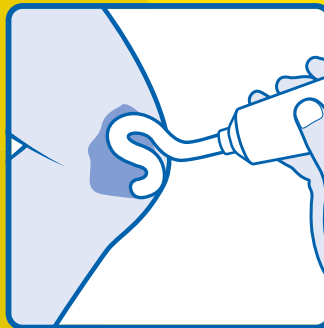


- ✓ Apply a layer of Flamigel® 1 – 2 times/day.
- ✓ For sunburn, apply Flamigel® 2 – 6 times/day.

② DRYING

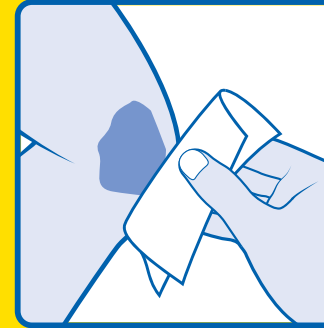

Gently dry the wound with a clean dry towel.

④ COVER

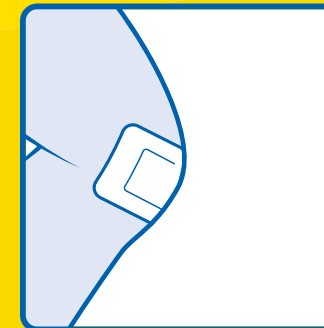

If necessary, cover the wound with a plaster or dressing and change daily.

If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days, seek medical advice.

Flamigel®

Flamigel®'s hydro-active colloid formulation helps absorb excess moisture for wet wounds or rehydrates dry wounds. As a result, Flamigel® ensures optimal moisture balance that promotes **40% faster healing***.

*Reference:

Thomas, S. Wound management and dressings. The pharmaceutical press London. 1990, Chapter 2: Functions of a wound dressing, p9-19.